

Bone Remove Kit

STABLE

SAFE

SIMPLE

EASY


FAST



ID Indicate different!
Dental

Room No. 501, 164, Obong-ro, Buk-gu, Daegu, Republic of Korea
TEL +82-53-381-9155 | FAX +82-53-215-2255


www.iddental4u.com

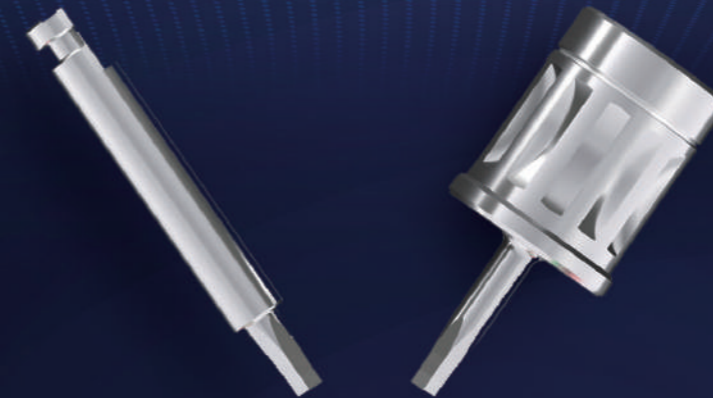

STABLE BONE REMOVER®

Scratch
Bone Heating
Damage

NO

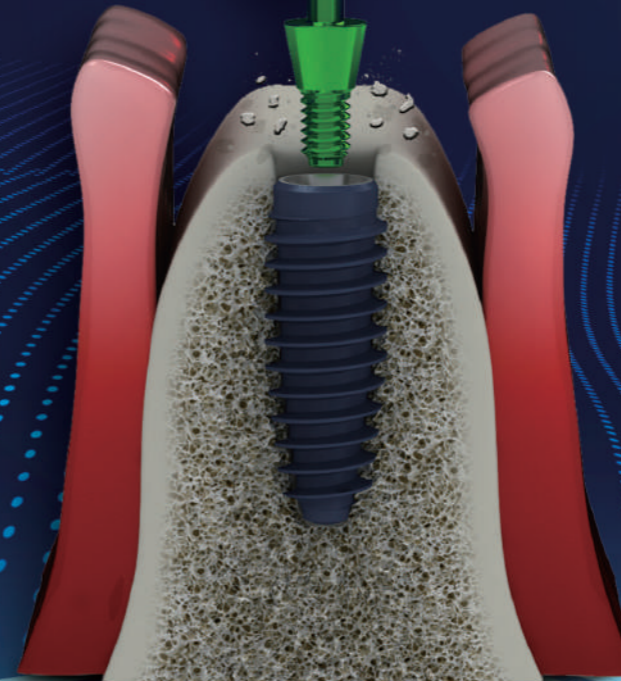


STABLE BONE REMOVER KIT

Core Driver




Core Pin

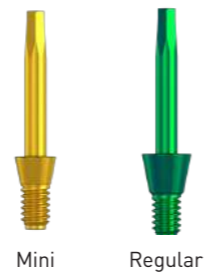

Remover Drill Driver




Remover Drill

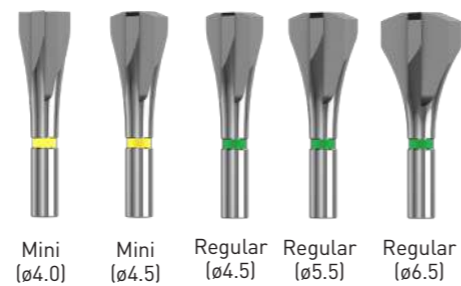

DRILLING SEQUENCE

Handpiece Type

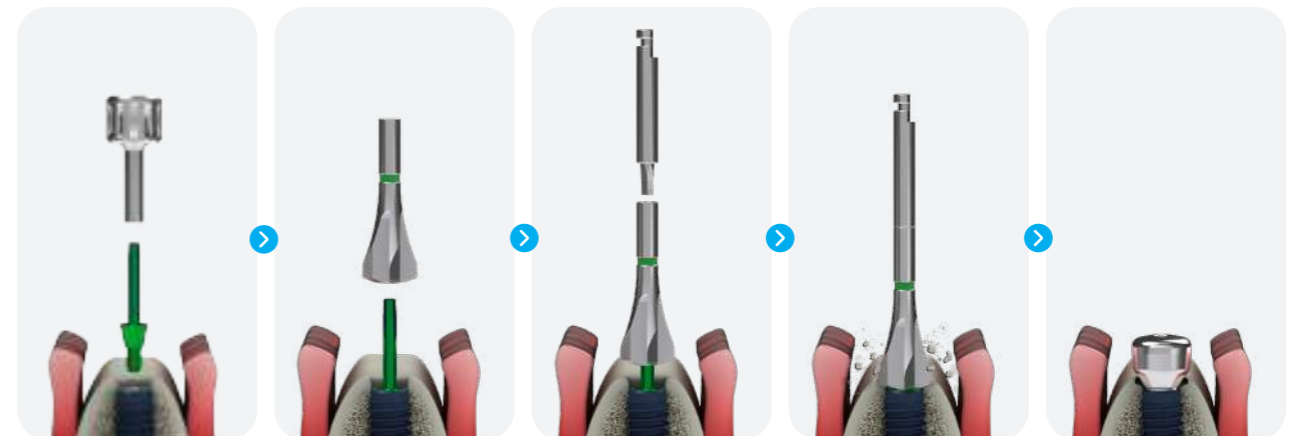

Connect the CorePin to the fixture with the Core Pin Driver


Insert the Stable Bone Remover Drill suitable for the size into the fastened Core Pin.

Fasten the 1.2 Hex Hand Piece Driver to the top of the inserted Stable Bone Remover Drill.

Remove ExcessBone slowly with torque 45rpm.

After removing the Core Pin, connect the Healing Abutment or Final Abutment suitable for the size to finish.

Handle Ratchet Type

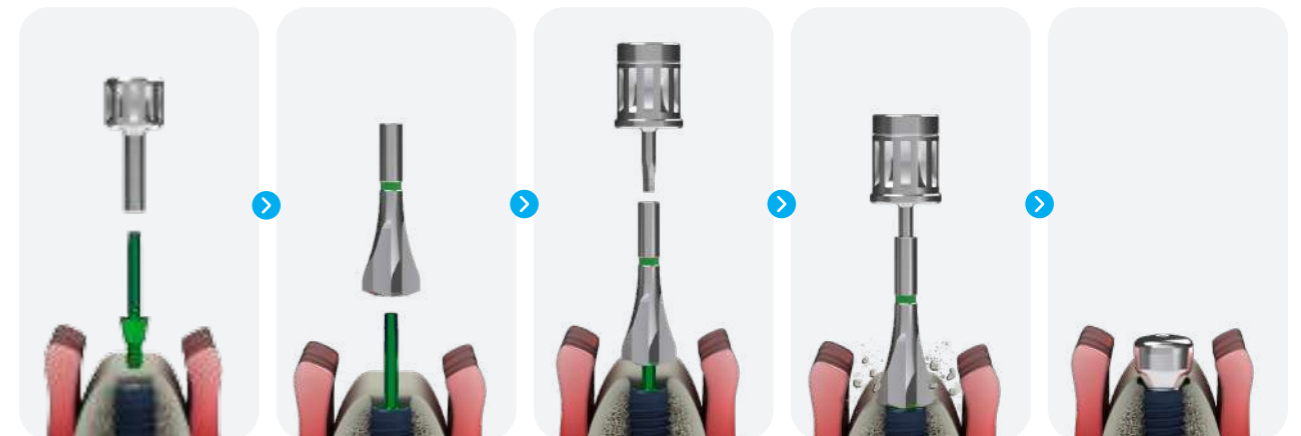

Connect the CorePin to the fixture with the Core Pin Driver.

Insert the Stable Bone Remover Drill suitable for the size into the fastened Core Pin.

Fasten the handle Driver to the top of the inserted Stable Bone Remover Drill.

Remove ExcessBone slowly with torque 35N .

After removing the Core Pin, connect the Healing Abutment or Final Abutment suitable for the size to finish.